

EASYGRIP[®]

TECHNOLOGY

► **try and adopt it!** ◀

Your doctor prescribed you a new treatment with a preservative-free formulation, delivered by EasyGrip[®] Technology

EasyGrip[®] Technology includes a multi dose bottle with a pump and a disposer system helper patented by Théa.

This innovation:

- **Delivers preservative-free treatment**
- **Is safe**
- **Is easy to use, thanks to:**
 - the easy opening
 - the natural position of the hand
 - 2 grips, which help you to push on the device
- **Allows a calibrated dose in every drop.**

You might get used to it, but try it and you will adopt it !



How to use **EASYGRIP**[®]

TECHNOLOGY

1 Wash your hands.



The first time, **unscrew the tamper-proof cap** to open the bottle. Before each use,

remove the cap from the bottle tip. Press down

several times with the bottle upside down, to activate the pumping mechanism until the first drop appears. **This process is only for the very first use and will not be necessary for the next administrations.**

2



3



Place:

- Your index finger on the base of the bottle.
- Your middle finger on the tab at the base of the bottle.
- Your thumb on the tab of the top of the bottle.

Hold the bottle upside down.

4



Tilt your head back a little, pull the lower eyelid down slightly. Hold the bottle dropper vertically above your eye.

Press briefly but firmly on the bottle.

Only one drop is released.

Close your eyelids and **press the inner corner of the eye** near your nose with your finger.

5



Close the tip of the bottle with the cap immediately after use.